# FLEXISTONE® Plus

**GB**

Instructions for use

*Insulating silicone material for use in dental technology, addition curing, low viscosity, high final hardness*

## Indications for use:

- Insulating material, A-silicone based, for the application with injecting technique and flask pressing technique. For the insulation and protection of acrylic or porcelain teeth during investing.
- Can be easily applied in one single procedure onto acrylic teeth and wax, forming a thin, very hard insulating layer for fixing of teeth during pressing.

Fig. 1

### 1. Dosage and mixing

Fill mixing bowl with base to the first marking, then with catalyst to the second one (Fig. 1). Spatulate **FLEXISTONE® Plus** homogeneously, until a uniform colour is achieved (Fig. 2).

Fig. 2

### 2. Application

Beginning from the interdental spaces, apply mixed **FLEXISTONE® Plus** uniformly with a spatula reaching a thickness of approximately 3 mm. Prior to setting of **FLEXISTONE® Plus** provide undercuts by means of a wax-knife (Fig. 3). Thus an optimum bonding to the plaster is ensured. No retention crystals are necessary.

Fig. 3

### 3. Deflasking

After polymerisation deflask as usual and remove **FLEXISTONE® Plus**.


## Important working hints

- Avoid thin-tapered borders of **FLEXISTONE® Plus** – cutting of a small shoulder is recommended to prevent separation in the flask-counter.
- Increased temperatures accelerate, decreased temperatures retard the setting time.
- Containers should be tightly closed immediately after each use!
- Cured materials are chemically inert – spots on clothing should be avoided!
- **FLEXISTONE® Plus** is recommended for the IVOCLAR SR-Ivocap-System.

## Technical data:

- **Mixing ratio:** 1:1
- **Colour code:** **FLEXISTONE® Plus** base: clear blue **FLEXISTONE® Plus** catalyst: white
- **Mixing time:** approx. 45 sec.
- **Working time:** approx. 4 min.\*
- **Setting time:** approx. 15 min.\*
- **Final hardness:** 75 Shore A
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F ± 4 °F, 50 ± 5 % rel. humidity

\* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5% rel. humidity. Increased temperatures accelerate, decreased temperatures retard the indicated times.

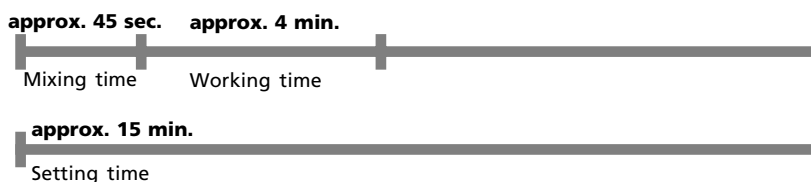## Ordering information:

**FLEXISTONE® Plus**  
Standard packing **02383**  
2 Plastic bottles, ea. 160 ml  
base + catalyst 1:1  
2 Stands  
1 Mixing bowl

Eco-packing **02329**  
2 Plastic bottles, ea. 1000 ml  
base + catalyst 1:1  
4 Mixing bowls

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