silasoft® direct

GB
Instructions for use

**Precision impression material, C-silicone based,
low consistency – light bodied – normal setting, 10:1**

Fig. 1

1. Preparation of the impression

Depending on the impression technique select an appropriate tray. For optimal adhesion of the impression to the tray apply a thin layer of **sili Adhesive Spray** or **sili Adhesive** liquid onto the tray and let dry for 60 sec (Fig. 1). Take preliminary impression with **silaplast FUTUR**. To facilitate the flowing out of excessive light body material from the impression cutting of escape vents and removal of interdental septi in the impression with the use of the special instrument **Deta-Cut** is recommended.

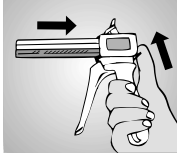
Fig. 2

2. Mixing and Dosing

Extrusion of the impression material from the cartridges by means of Automix2 gun (Fig. 2). Place cartridge into mixing gun, remove cartridge plug by rotation. Eject a small amount of material (bleeding) to control that equal amounts are being extruded from both orifices (Fig. 3). Insert mixing cannula into guide grooves on the cartridge and rotate anti-clockwise to a stop. Squeeze mixing gun handle for individual dosage (Fig. 4). For application into sulcus area use Intra-Oral tips yellow – coloured or a disposable syringe (Fig. 5). After use remove mixing cannula and close the cartridge by means of the original closure cap.

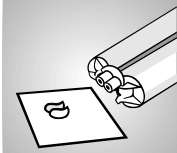
Fig. 3

3. Disinfection

After removal rinse impression under lukewarm water. Disinfection for 15 min. in 2 % glutaraldehyde.

Fig. 4

4. Model casting

Not before 30 min. after removal from the mouth, at the latest after 24 hours. Recommended materials: Dental plasters of class III and IV-also commercially available model acrylics.

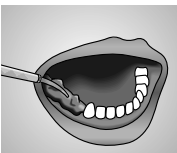
Fig. 5

5. Galvanisation

Impressions can be electroplated with commercially available copper and silver plating baths.

6. Cleaning of trays

Remove set material with a blunt instrument. **sili Spray** or **sili liquid** film can be easily removed with **sili Spray Solvent** (free of FCHC). Work in well ventilated rooms. Clean and disinfect as usual.


Important working hints

- Increased temperatures accelerate, decreased temperatures retard the setting time.
- After use reclose cartridge orifices carefully.
- Do not intermix with addition curing silicones.
- Avoid staining of clothing. Cured impression materials cannot be removed.
- Remove all residues of impression material from the patient's mouth.
- Avoid contact of catalyst with eyes, may cause irritation. If necessary flush eyes with plenty of water and seek medical care.

Other information:

Silicone impression materials are proven a million times over, unwanted effects cannot be expected in proper use. However, immune reactions, e.g. allergies or irritations cannot be excluded basically. In the case of doubt, we recommend performing an allergy test before application.

For use by trained specialists.

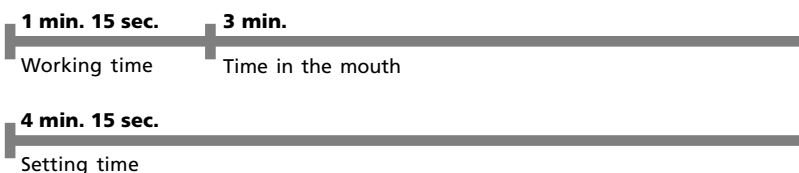Indications for use:

- Preliminary impressions for the double impression or double mix technique (wash)
- Reline impressions

Technical Data:

DIN EN ISO 4823 – Type 3

- **Mixed volume:** 50 ml (cartridge)
- **Mixing ratio:** 10:1
- **Colour code:** base: violet catalyst: blue
- **Mixing time:** omitted (Automix2-system)
- **Working time:** approx. 1 min. 15 sec.*
- **Setting time:** approx. 4 min. 15 sec.*
- **Time in mouth:** approx. 3 min.
- **Strain in compression:** approx. 5 %
- **Recovery from deformation:** 98.50 %
- **Linear dimensional change:** 0.50 %
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

Ordering information:

silasoft® direct Automix2
Standard packing **02809**
2 x 50 ml cartridges
12 mixing cannulas,
yellow 10:1

Eco packing **02810**
8 x 50 ml cartridges
48 mixing cannulas,
yellow 10:1
48 Intra-Oral-Tips, yellow

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silaplast

FUTUR

high final hardness
normal setting

Preliminary impression
material for corrections of
preparations, tray material,
overimpressions, wax-bite

Wash
material
light bodied

low viscosity
low con-
sistency

silasoft® N

normal setting

Correction impression
material for the double
impression and double
mix technique, relinings

Wash
material
light bodied

low viscosity
low con-
sistency

silasoft® S

normal setting

Correction impression
material for the double
impression and double
mix technique, relinings

Functional
heavy bodied

high con-
sistency and
plasticity

sta-seal f

medium final hardness
plastic phase
extended setting time

Functional impressions,
functional border moulding

Catalyst
paste

Catalyst
liquid

cat^p / cat^f

FUTUR

inert system
non-irritating

cat^p universal paste,
cat^f silaplast liquid,
silasoft® cat^f, liquid,

Special
instrument

Accessory

Deta-Cut

sili adhesive

sili solvent

Sterilizable, curved, for
cutting the preliminary
impression

adhesive for C-silicones
removal of the adhesive
and cleaning of trays

All DETAX condensation-
curing silicones are
portioned with
dispensing aids (tray,
measuring bowl,
measuring scale etc.) and
kneaded by hand or
mixed homogeneously on
a mixing pad.

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