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**DENTURE CLEANER
PROTESERENSEMIDDEL**

200 g (7 oz)

RENEW®
CANADA

REMOVES: Tobacco, coffee, tea, iron, calcium, calculus, plaque, tartar. Use 1 teaspoon of powder per cup of warm water. Stir solution thoroughly in a denture bath. **IMMERSE DENTAL APPLIANCE FOR 20 MINUTES.** Extremely stained dentures or dentures with heavy build-up of calculus and tartar may require overnight soaking. Do not soak metal partial dentures for more than 20 minutes. **IMPORTANT:** Dentures with soft liner should not be soaked for more than 20 minutes per soaking. **SAFETY CAUTION:** Renew is not to be taken internally. If swallowed do not induce vomiting, give large quantities of water. Call physician immediately. In case of contact with eyes or mouth flush with water.

ALLERGIC REACTION: As with any denture cleaner an allergic reaction can occur. If an allergic reaction occurs, immediately discontinue use. **RINSE THE DENTAL APPLIANCE** thoroughly before using. **KEEP OUT OF REACH OF CHILDREN.** **DO NOT** use to clean jewellery. **DO NOT** mix with other chemicals. **AVOID** spilling on clothing, fabrics and countertops.

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Made in Canada/Made in Canada Manufactured by/fremstillet af

MID-CONTINENTAL
DENTAL SUPPLY CO. LTD.

Headingley, MB R4J 1A4 • Ph: 1-800-882-7341
www.mid-continental.com

6 26103 00001 5