Primo

GB

Instructions for use

Adhesive for Molloplast® B-relinings

By use of **Primo adhesive** the chemical bonding of **Molloplast® B** to the acrylic denture base is achieved. However, the adhesive is only applied onto already polymerised denture base material. Prior to the application of **Primo adhesive** the denture

base is prepared according to **Molloplast® B** instructions for use.

Then **Primo adhesive** is brushed uniformly 1–2 times onto the clean and dried denture base. After reacting for approx. 60–90 minutes **Molloplast® B** is pressed onto the denture base and polymerised according to instructions.

Important working hints

- **Primo** coating must cover the entire surface of denture acrylic in contact with **Molloplast® B**. After a reacting time of approx. 60–90 minutes **Molloplast® B** is pressed onto the denture base and polymerised according to instructions.
- Close bottle tightly after each use.
- Avoid contact with eyes, mucous membranes and skin.
- Do not breathe vapours.

Indications for use:

- **Primo adhesive** ensures an optimum bonding of **Molloplast® B** to polymerised acrylics.

Technical data:

- **Volume:** 15 ml, vial with integrated brush
- **Drying time:** 2–3 minutes
- **Reacting time:** 60–90 minutes
- **Storage:** Keep container tightly closed and at a temperature not exceeding 25 °C / 77 °F. Keep away from moisture.

Ordering information:

Primo adhesive Standard
packing 03004
Bottle of 15 ml

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